

# CAPGUARD

Extra Protection for Employees and Customers  
while Branding your Business



**Our clear PETG plastic  
Cap Guard lens attaches  
to the brim of any  
standard baseball hat or  
visor with 3 patent  
pending clips.**

## FEATURES:

- Additional layer of protection against Covid-19
- Easily secures to uniform cap maintaining company branding
- Clean non-medical appearance
- Re-usable, comfortable and lightweight
- Unobstructed view while allowing for clear communication
- Constructed from top quality FDA-approved material
- Made in USA – providing jobs and supporting our economy

Call us today to Place your Order! (310) 608-1115 or  
Visit us Online at [www.pvctech.com](http://www.pvctech.com)

## Assembly Instructions – Cap Guard

Thank you purchasing your new Cap Guard and helping to fight against the spread of the COVID 19 virus.

We have created this simple Step-by-Step guide to help you achieve the desired fit. Please review and follow the steps below:

- Start by removing the two protective layers (1) from your lens (2).
- Ensure your ball cap or visor brim has a standard brim curvature (3) to aid the fit. The radius does not need to be exact as the shield assembly will hold the brim shape after installation.
- Slide the Black center clip (4) onto the lens aligning it with the center notch, then the two Grey side clips (5) aligning them with the side notches.
- Now slide the center clip with the lens assembly onto the center of your brim to secure the lens in place.
- With the lens now securely held by the center clip, you can rotate the two Grey side clips onto the brim as shown.
- The basic assembly is complete; however, the clips will often require minor adjustment to achieve a smooth lens radius.
- Simply adjust the position of the clips on the brim until you achieve a consistent lens radius.
- That's it! You are done and ready to enjoy your new Cap Guard!

**If you have any questions or concerns, please do not hesitate to contact us at [angie@pvctech.com](mailto:angie@pvctech.com) 310-900-7785**

